



ApneaWear.ca

## Theme Hose Covers

100% Cotton (*flannel*)



Place a small piece of plastic wrap (*not included*) over the smallest end of your tubing.


The plastic wrap helps to cover the small rubber end of the tube and make it slide effortlessly through the hose cover.



Place the tube over your shoulder and drop the small rubber end through the hose cover.



You will need to guide the remaining fabric at the end through the tube.

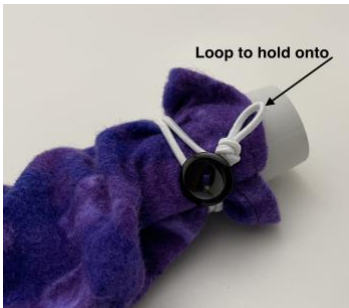

Secure the ends with the button and elastic loop closure.

There is additional fabric to allow the tube to bend and stretch.

*Machine wash cold, tumble dry low. Remove promptly. Do not use bleach.*